
TWAIN Trium 2D INSTALLATION GUIDE



Ed. n. 1	Rev. n. 0	Issued by: SM	Reviewed by:	p. 1/20
	of: 2017-06-15	Approved by:		

DETAILS OF THE DOCUMENT

Project Name: TWAIN Trium 2D

File name: TWAIN-TRIUM_installation_guide.docx

Document type: Technical specification

Standard reference: N.A.

EDITION	REVISION	DATE	ISSUED BY	REVIEWED BY	APPROVED BY	NOTES
1	0	2017/06/15	M. SASSO			FIRST RELEASE
1	1	2017/07/05	M.SASSO			REMOVE INSTALLATION OF VCREDIST CHANGE "SETUP_TWAIN_TRIUM" CHANGE PATH FOR "TRIUM_CONFIG_INSTALLER"

Language of the original document: English



THE INFORMATION CONTAINED IN THIS DOCUMENT ARE STRICTLY CONFIDENTIAL AND CAN ONLY BE VIEWED BY AUTHORIZED PERSONNEL OF ACTEON Group.

ANY INFORMATION AND DATA CONTAINED HEREIN IS EXCLUSIVE PROPERTY OF ACTEON AND ANY DISCLOSURE, COPYING, DISTRIBUTION OR TAKING ANY ACTION IN RELIANCE ON THE CONTENTS OF THIS DOCUMENT IS STRICTLY PROHIBITED UNDER ANY CIRCUMSTANCE AND MODALITY.

Ed. n. 1	Rev. n. 0	Issued by: SM	Reviewed by:	p. 2/20
	of: 2017-06-15	Approved by:		

	INSTALLATION GUIDE	TWAIN- TRIAM_installation_guide1.1.docx
Via Roma, 45 21057 Olgiate Olona (VA)	Project Name: TWAIN Trium 2D	Page 3 of 20

Summary

Quick description of the TWAIN standard	4
Main Description	4
Objective.....	4
INTRODUCTION	5
CONTENT OF THE USB STICK / DVD ROM PROVIDED WITH TRIUM	5
WORKSTATION MINIMUM REQUIREMENTS	6
INSTALLING THE TWAIN INTERFACE FOR XMIND-TRIUM 2D	8
DCAM INSTALLATION.....	11
NETWORK CARD INTEL CT CONFIGURATION	13
TRIUM INTERNAL BOARDS FIRMWARE UPGRADE	14
TRIUM VARIABLE CONFIGURATION SERVICE TOOL	17
RX TUBEHEAD CALIBRATION	19
INTRODUCTION.....	19
PROCEDURE.....	19
LOGIN to SERVICE	19
RX TUBE CALIBRATION	19

Ed. n. 1	Rev. n. 0	Issued by: SM	Reviewed by:	p. 3/20
	of: 2017-06-15	Approved by:		

	INSTALLATION GUIDE	TWAIN- <i>TRIUM_installation_guide1.1.docx</i>
Via Roma, 45 21057 Olgiate Olona (VA)	Project Name: TWAIN Trium 2D	Page 4 of 20

Quick description of the TWAIN standard

About the TWAIN consortium:

The TWAIN Working Group is a not-for-profit organization which represents the imaging industry. TWAIN's purpose is to provide and foster a universal public standard which links applications and image acquisition devices. The ongoing mission of this organization is to continue to enhance the standard to accommodate future technologies.

The TWAIN consortium does not market any software, hardware or other products. It has a website at <http://www.twain.org/> on which you can find a wealth of practical information for people who want further information on the subject.

The TWAIN standard exists for Windows and Macintosh platforms. Only Windows is covered herein. At the present time, there is no Macintosh version of the TWAIN interface for SOPIX sensors.

The TWAIN standard allows total interconnection of software and peripherals. However, software and peripherals are not obliged to comply with the entire standard; so there can be partial or complete incompatibilities in some cases.

Main Description

Objective

Acquire TRIUM PANORAMIC & CEPHALOMETRIC images through TWAIN Source provided by Acteon.

The configuration of TRIUM managed are:

- PANORAMIC
- PANORAMIC/CEPHALOMETRIC

Ed. n. 1	Rev. n. 0	Issued by: SM	Reviewed by:	p. 4/20
	of: 2017-06-15	Approved by:		

	INSTALLATION GUIDE	TWAIN- TRIAM_installation_guide1.1.docx
Via Roma, 45 21057 Olgiate Olona (VA)	Project Name: TWAIN Trium 2D	Page 5 of 20

INTRODUCTION

This document contains the setup and the configuration of the Workstation with for Trium Pan /Ceph tomograph.

This manual is intended only for new installation of the ais 1.6 software on a brand new x-mind trium, and not for the upgrade of a previous version.

Please follow all the instruction included in this manual to correctly install the new version of the software.

Changing some options in the installation could affect the right functionality of ais software, generating some faults.

CONTENT OF THE USB STICK / DVD ROM PROVIDED WITH TRIUM

1. DOCUMENTATION
2. Setup TWAIN TRIUM
3. NEW HAMMAMATSU DCAM SETUP
4. TRIUM CONFIG INSTALLER

Ed. n. 1	Rev. n. 0	Issued by: SM	Reviewed by:	p. 5/20
	of: 2017-06-15	Approved by:		

	INSTALLATION GUIDE	TWAIN- TRIUM_installation_guide1.1.docx
Via Roma, 45 21057 Olgiate Olona (VA)	Project Name: TWAIN Trium 2D	Page 6 of 20

WORKSTATION MINIMUM REQUIREMENTS

IT'S MANDATORY TO RESPECT WORKSTATION MINIMUM REQUIREMENTS



THE CHOICE OF WORKSTATION/SERVER FOR X-MIND TRIUM PANO/CEPH, OR OTHER VERSIONS OF TRIUM PROVIDED WITHOUT WORKSTATION, IS UNDER CUSTOMER RESPONSIBILITY. A WRONG CHOICE, WITHOUT RESPECTING MINIMUM REQUIREMENT, WILL NOT BE UNDER ACTEON GROUP RESPONSIBILITY.

Please, take care about workstation minimum requirement to avoid faults, errors, misfunctionality and/or problem during the installation, connection and use of X-MIND Trium.

WORKSTATION:

- MAC Workstations are not supported
- Virtual machines installation is not tested and not permitted

OPERATIVE SYSTEM :

- Microsoft Windows 7 64 bits
- Microsoft Windows 10 professional 64 bits

HARDWARE REQUIREMENTS :

- Processor : INTEL I5 or superior
- RAM : 4GB DDR3 1600MHz in Dual Channel configuration
- Hard Disk : 1TB 7200 rpm or superior
- Network Cards :

2 LAN network boards needed : one of them can be already present on the motherboard and have being used only for network access. The second one MUST be and INTEL CT 1000 pro (provided with Trium PAN/CEPH) and MUST be used for the connection between Trium and workstation

Avoid to use a wireless network card on the workstation : fast connection network (LAN type) are needed to share the patient exams with the clients PCs. Due to this, a WiFi network card could be too slow to transfer patient exams with clients PCs.

- Graphic card : OPEN GL 2.1 compatible (suggested an NVIDIA GT/GTX)
- Monitor : 1280x1024 16 million of colors at least. HD resolution is suggested.

ADMINISTRATION RIGHTS :

Ed. n. 1	Rev. n. 0	Issued by: SM	Reviewed by:	p. 6/20
	of: 2017-06-15	Approved by:		

	INSTALLATION GUIDE	<i>TWAIN-TRIUM_installation_guide1.1.docx</i>
Via Roma, 45 21057 Olgiate Olona (VA)	Project Name: TWAIN Trium 2D	Page 7 of 20

- **Always take care to be the administrator of the workstation, and log in as the same user of the AIS installation.**



Ed. n. 1	Rev. n. 0	Issued by: SM	Reviewed by:	p. 7/20
	of: 2017-06-15	Approved by:		

	INSTALLATION GUIDE	<i>TWAIN-TRIUM_installation_guide1.1.docx</i>
Via Roma, 45 21057 Olgiate Olona (VA)	Project Name: TWAIN Trium 2D	Page 8 of 20




INSTALLING THE TWAIN INTERFACE FOR XMIND-TRIUM 2D




Installation is made as simple as possible.

You just need to run the file **TriumTwainDriverSetup_x.x.x-n.exe**, and installation takes only a few seconds. Follow the steps until message informs you when the installation is finished.

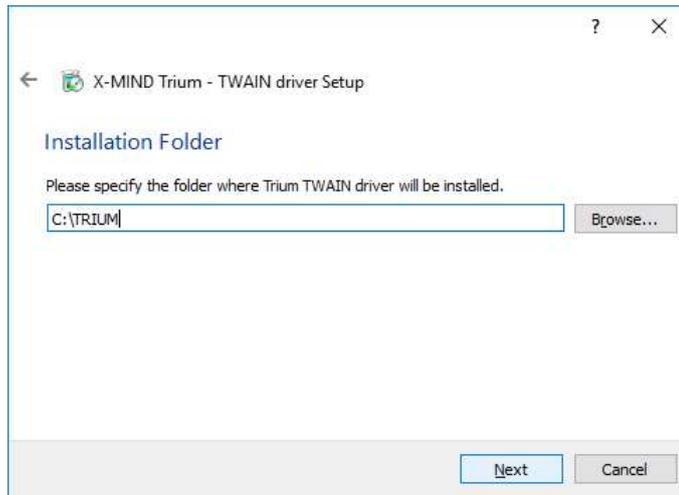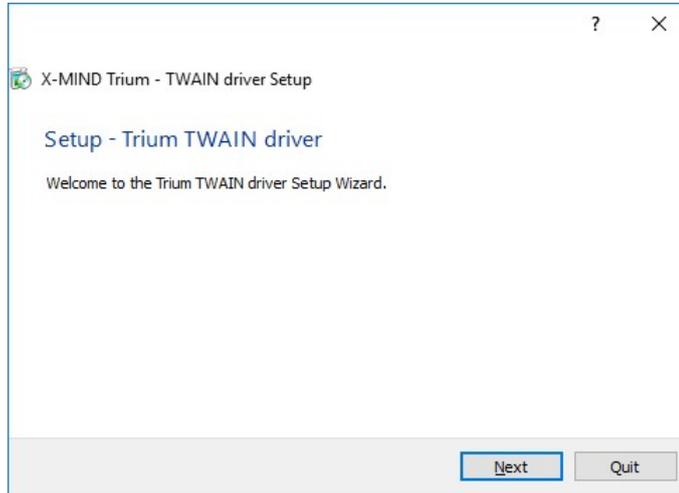

Ed. n. 1	Rev. n. 0	Issued by: SM	Reviewed by:	p. 8/20
	of: 2017-06-15	Approved by:		

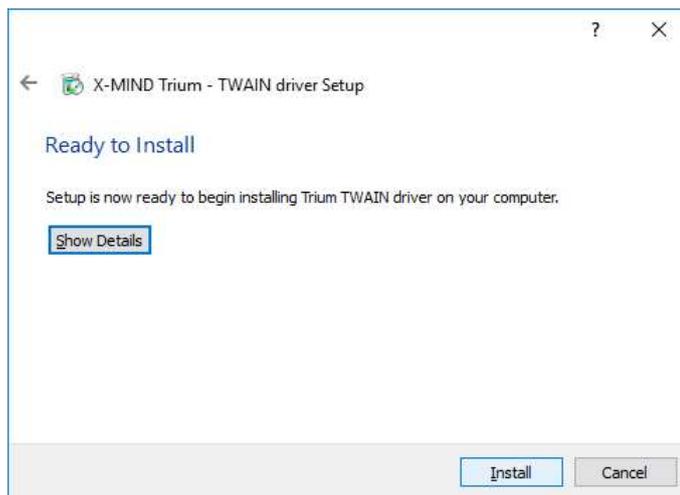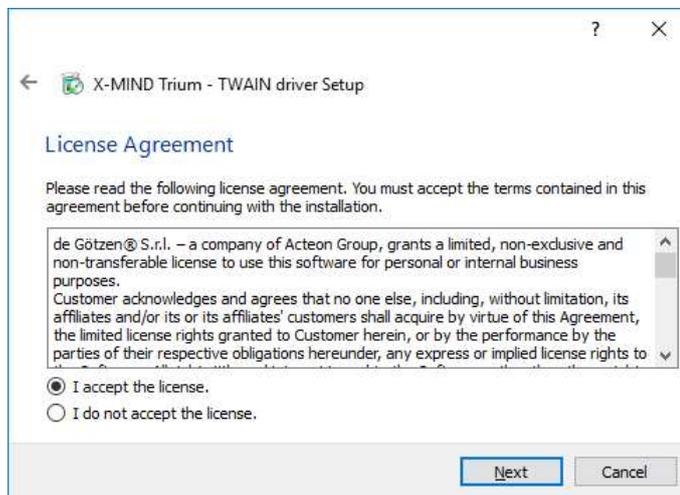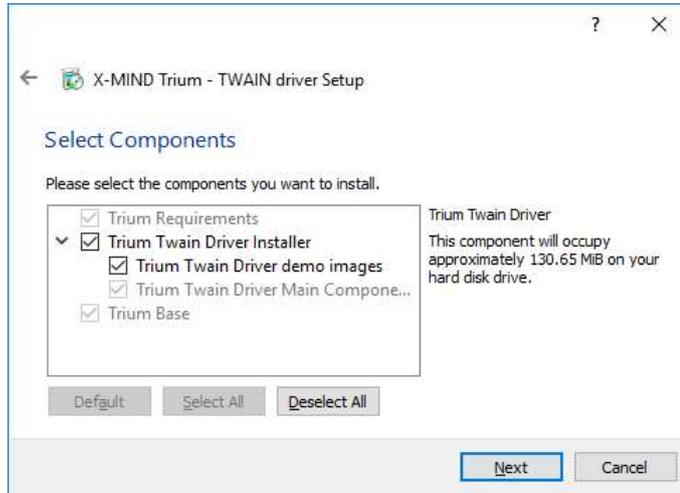


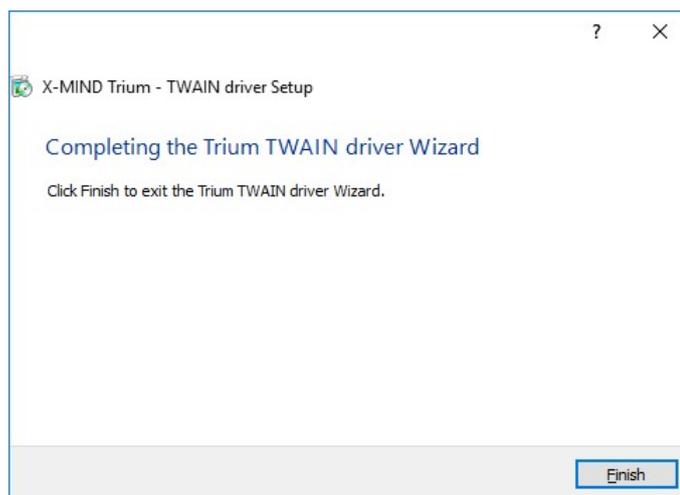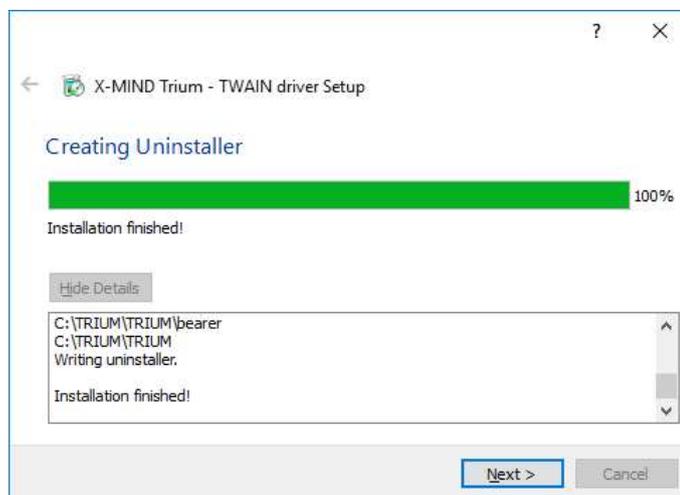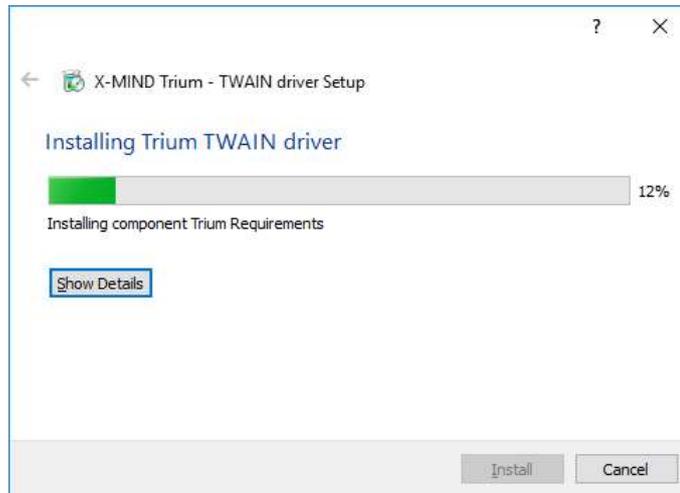



Ed. n. 1	Rev. n. 0	Issued by: SM	Reviewed by:	p. 10/20
	of: 2017-06-15	Approved by:		

DCAM INSTALLATION

1. Install the new DCAM as following:

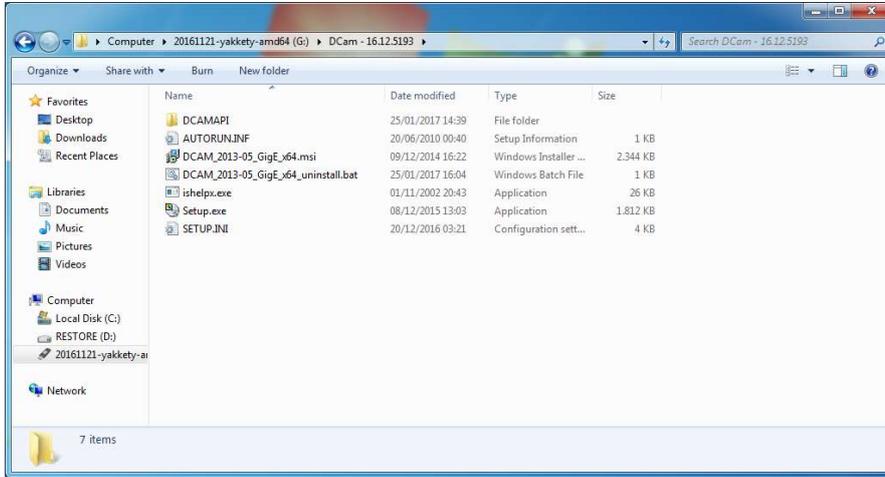


- Launch from the provided **USB stick/zip\DCam - 16.12.5193** folder file the SETUP executable
- the setup application will allow you to choose the installation type: choose GigE Camera for Flat Panel Sensor, then click on install, accept the license and click on next

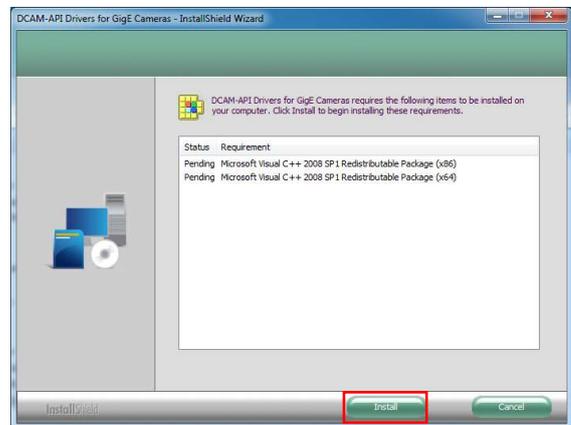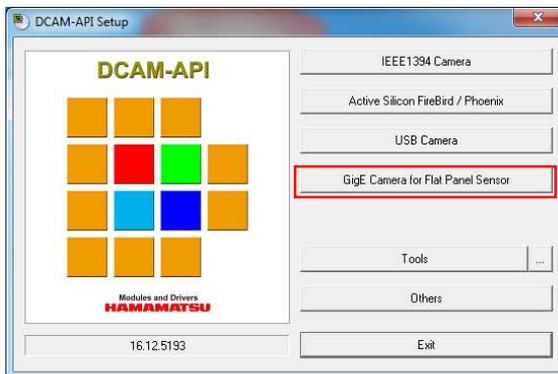


Ed. n. 1	Rev. n. 0	Issued by: SM	Reviewed by:	p. 11/20
	of: 2017-06-15	Approved by:		

	INSTALLATION GUIDE	TWAIN-TRIUM_installation_guide1.1.docx
Via Roma, 45 21057 Olgiate Olona (VA)	Project Name: TWAIN Trium 2D	Page 12 of 20

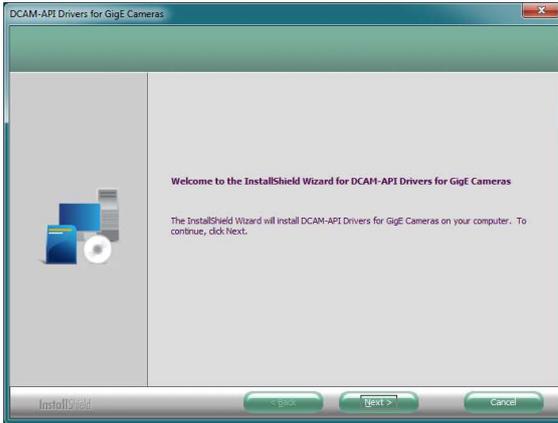


- when the installation is finished, you don't need to set the configuration of the driver

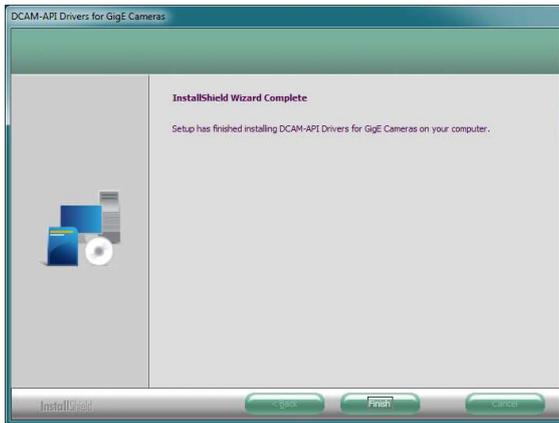

Ed. n. 1	Rev. n. 0	Issued by: SM	Reviewed by:	p. 12/20
	of: 2017-06-15	Approved by:		

NETWORK CARD INTEL CT CONFIGURATION

1. Install from the provided USB stick the INTEL CT network driver
2. From network connection properties set TCP/IP internet protocol with persistent IP address as follow:

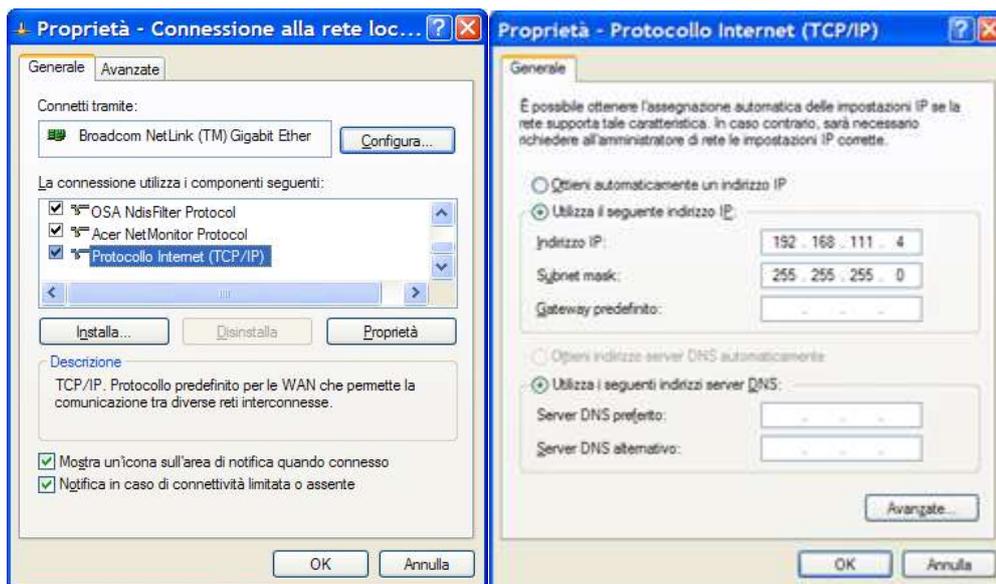

3. SET NIC IP address to 192.168.111.4
4. SET Subnet mask to 255.255.255.0.
5. Rename the NIC connection to the EUT as **“TRIUM – NOT TO MODIFY”**.



TRIUM INTERNAL BOARDS FIRMWARE UPGRADE

After the starting of TWAIN TRIUM, if the software detect a new firmware board available for any board, it will propose you, using the XM Driver. Follow step by step the instruction proposed. Don't change or do nothing else, otherwise the machine could be not functional. Firmware upgrade is under technician responsibility.

- During the firmware upgrade:
1. DON'T SWITCH OFF THE TRIUM
 2. DON'T SWITCH OFF THE WORKSTATION
 3. DON'T DISCONNECT THE LAN CABLE BETWEEN TRIUM AND THE WORKSTATION
 4. DON'T OPEN PROGRAM OR FOLDER IN THE WORKSTATION
- Making these operation will interrupt the correct firmware upgrade and can damage the Trium or put it out of order.
 These kind of operator's faults are under **operator responsibility**



1. Select the board and click on UPDATE button (Inverter board in this example)

FW UPDATE

INVERTER

UPDATE

INVERTER

Main MODE=1 VLD=1 VER=0.43.2 (C:0.43.2)

inverter MODE=2 VLD=0 VER=2.2.0 (C:2.2.0)

Power MODE=1 VLD=1 VER=0.9.2 (C:0.0.0)

Aux Cbct MODE=1 VLD=1 VER=0.7.0 (C:0.7.0)

INSTRUCTIONS


Use the following instructions when the upgrade of the FW of one or more XMtrium boards is required:

- 1) Select the board that requires FW upgrade;
- 2) Click on **UPDATE** button;
- 3) Wait until the two different progress bars are completed and appears the GREEN flag icon.




UPGRADE AUX CEPH ONLY IF CEPH IS PHYSICALLY INSTALLED ON TRIUM.
IF YOU UPDATE IT WITHOUT CEPH INSTALLED, THE UPGRADE FOR THIS BOARD WILL FAIL

2. Wait until the two different progress bars are completed and appears the GREEN flag icon

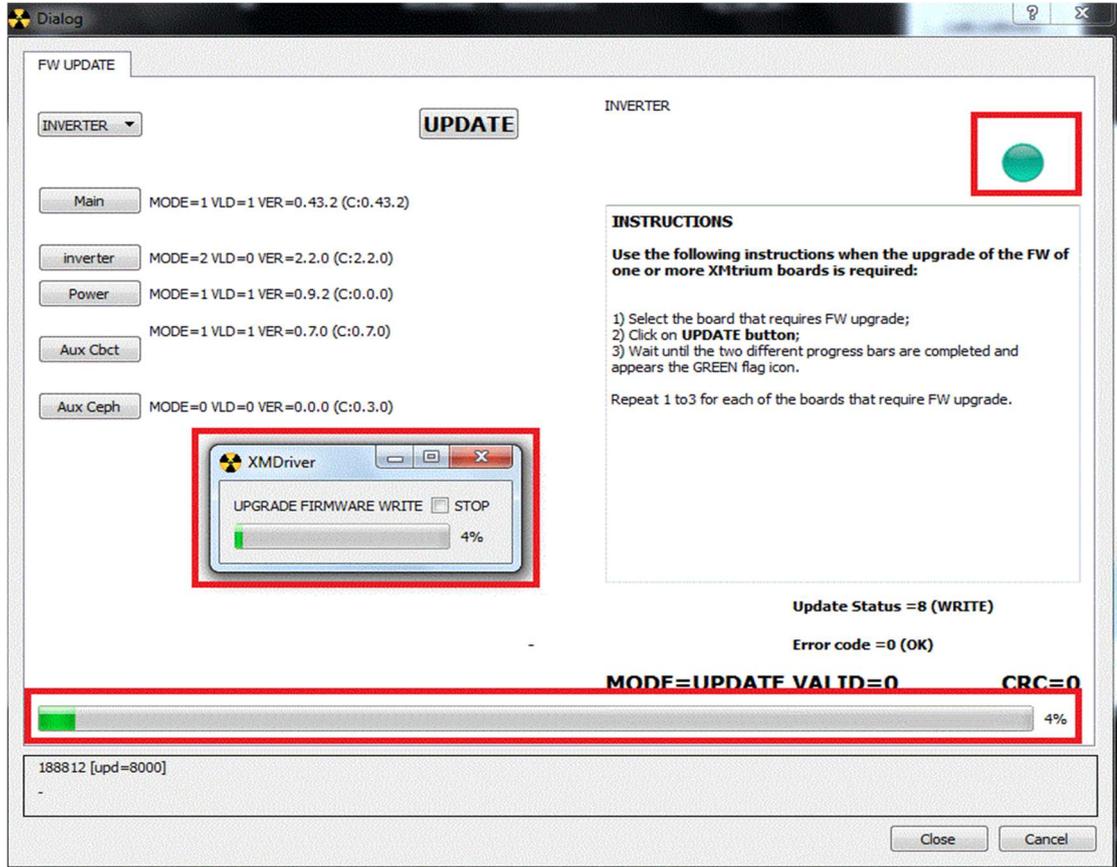

3. When the GREEN round icon stops blinking and becomes a flag, the upgrade is finished

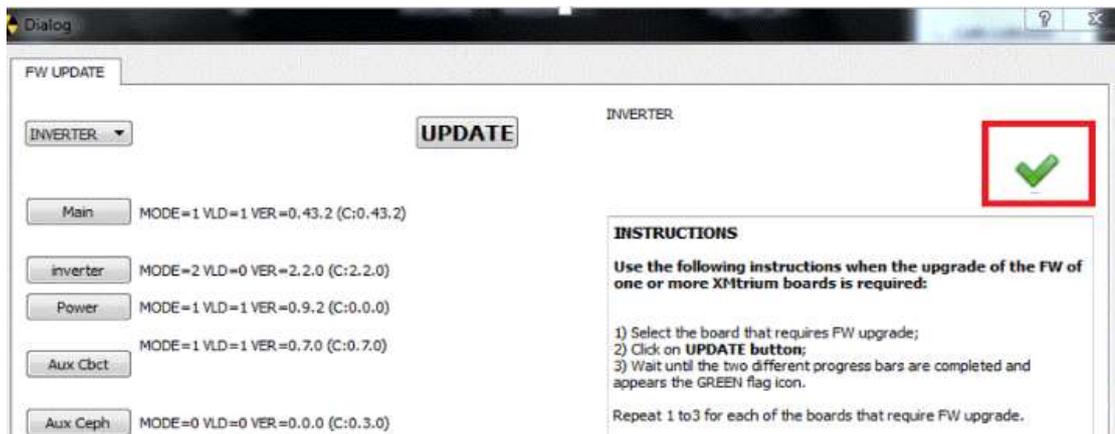

	INSTALLATION GUIDE	TWAIN- TRIMUM_installation_guide1.1.docx
Via Roma, 45 21057 Olgiate Olona (VA)	Project Name: TWAIN Trium 2D	Page 16 of 20

4. Make effective the modification:

✓ Turn OFF XM trium (do not use emergency stop to turn OFF trium) and **leave OFF at least:**

- 2 minutes after inverter board FW upgrade
- 30 seconds after other boards FW upgrade


For each board mounting old versions, upgrade the firmware, restarting from step 1 of this chapter

Ed. n. 1	Rev. n. 0	Issued by: SM	Reviewed by:	p. 16/20
	of: 2017-06-15	Approved by:		


TRIUM VARIABLE CONFIGURATION SERVICE TOOL

After the installation of AIS 1.6 run this tool to transfer and configure the Trium variable data setting in the workstation. This operation is mandatory having the Trium calibration data and factory test data inserted in the workstation. Without this data setting, the machine will not be functional.

1. Run TriumConfig_Installer_v1.4, double clicking on the setup file.
2. Click on 'Next' to proceed with the installation path :

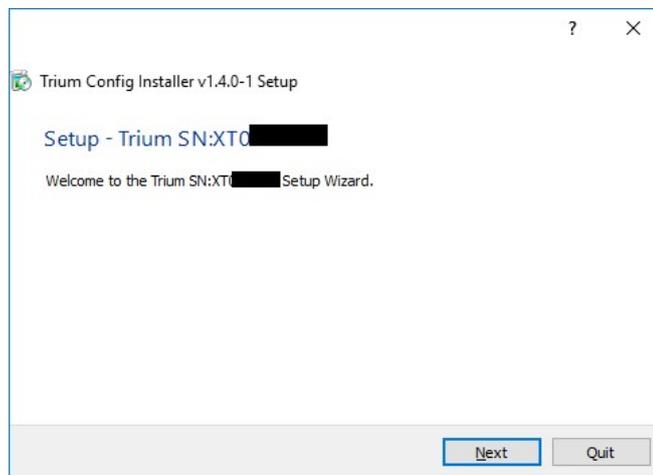

3. You MUST change the installation path to **C:\TRIUM**, or the machine will not be functional.

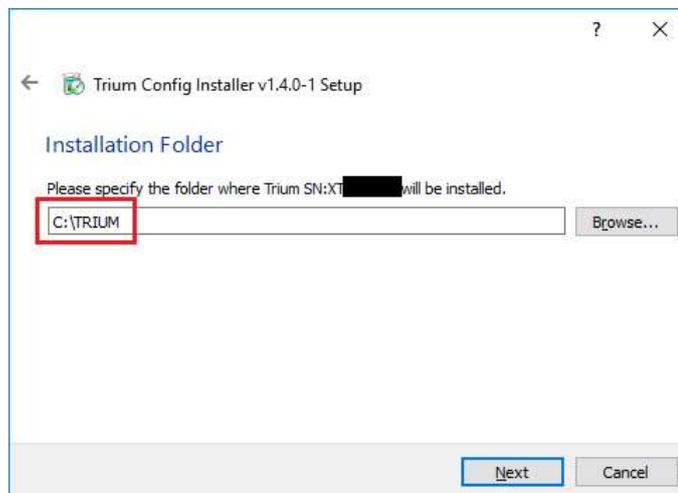

	INSTALLATION GUIDE	<i>TWAIN-TRIUM_installation_guide1.1.docx</i>
Via Roma, 45 21057 Olgiate Olona (VA)	Project Name: TWAIN Trium 2D	Page 18 of 20

4. Proceed with license agreement, the click on 'Next'
5. Click in 'Install' to start the installation
6. Click on 'Finish' to complete the installation.

Ed. n. 1	Rev. n. 0	Issued by: SM	Reviewed by:	p. 18/20
	of: 2017-06-15	Approved by:		

RX TUBEHEAD CALIBRATION

INTRODUCTION

In this paragraph you can find instructions to calibrate the RX tube.

These instructions are applicable to all the X-MIND Trium models.


Perform this calibration when:

- ✓ You replace the whole U-arm group
- ✓ You replace the RX tube
- ✓ You replace the Inverter board
- ✓ In some cases of RX tube failures, depending on troubleshooting diagnostic messages.

PROCEDURE


LOGIN to SERVICE

- a. End execution of AIS.
- b. Turn OFF then ON X-MIND trium.
- c. Start XMdriver Show double clicking on the file: C:\AISSoftware\XMdriver\XMdriverShow.bat
- d. Verify that SERVICE tab has green flag (WorkStation and X-MIND trium are communicating).
- e. Click on SERVICE tab and enter the required password: rdtech.

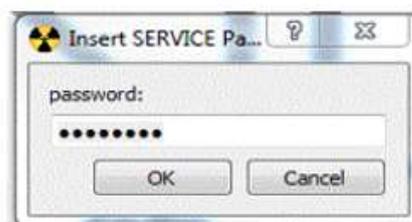


RX TUBE CALIBRATION

- f. Click on Calibration tab and then click on RX tube tab.

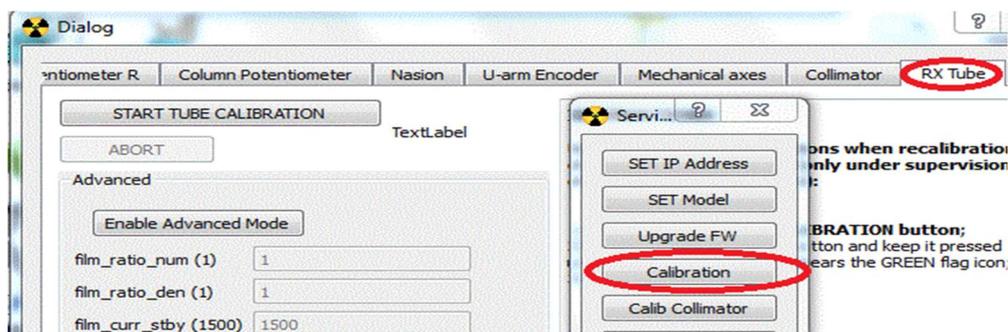

Ed. n. 1	Rev. n. 0	Issued by: SM	Reviewed by:	p. 19/20
	of: 2017-06-15	Approved by:		


g. Click on on START tube calibration.

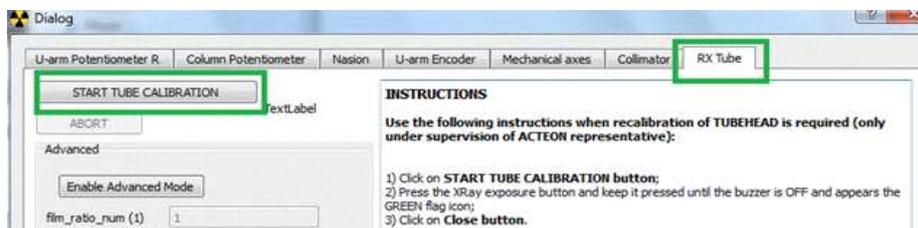

h. Keep continuously pressed the X-ray exposure button



until the buzzer is OFF and appears the green fixed icon.

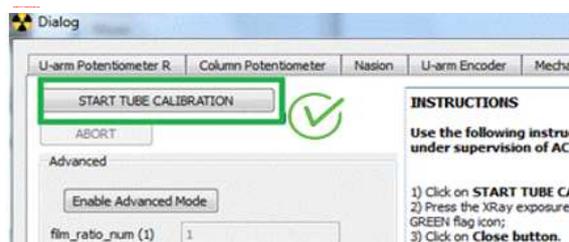

➔ **It can take up to 2 minutes: DO NOT INTERRUPT the procedure**

i. To make effective the modification turn OFF X-MIND trium.

j. End X-MIND driver SERVICE.

Ed. n. 1	Rev. n. 0	Issued by: SM	Reviewed by:	p. 20/20
	of: 2017-06-15	Approved by:		