



8 Westchester Plaza, Suite 112, Elmsford, NY 10523 • (914) 592-6100 • Fax (914) 592-6148

www.ImageWorksCorporation.com

INSTALLING TWAIN FOR YOUR EVA SELECT SENSOR.

Your EVA Select sensor is activated to interface with most imaging software using the TWAIN protocol. You will need to install the EVA Select TWAIN software and drivers onto the computers you will be connecting your sensor to.

Step Description

- 1 Log into your computer on an account with Admin privileges.
- 2 Set the UAC (User Account Control) to minimum for the installation.
- 3 Unplug the EVA SELECT from your computer.
- 4 Plug in the USB Thumb drive than was packaged with your sensor into your computer.
- 5 View the files on the thumb drive.
- 6 Double click on the "TWAINInstall.cmd" file to run the TWAIN Installer.
- 7 Observe and follow the instructions on the computer screen you may need to plug in, disconnect and reconnect the EVA Select Sensor during this installation.
- 8 After the installation is complete, connect your Image Application Software to the TWAIN Source "EVA SELECT". The specific steps are different for each application so you may need to consult your software's documentation.