


Sensibles Anterior Positioning Guide




Maxillary

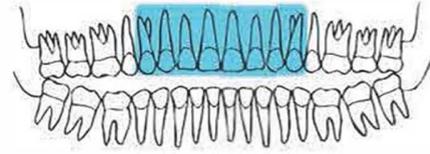


Use either Bite Block, depending on size of sensor.



Use Alignment Arm "ANTE 40857-10."

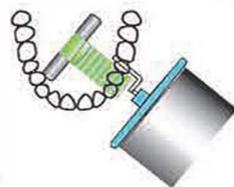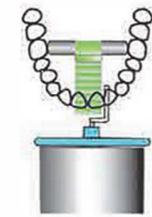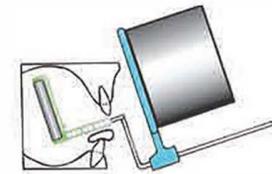


Mandibular

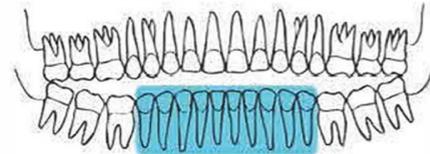

Use Alignment Arm "ANT#0857-10."





Use either Bite Block, depending on size of sensor.

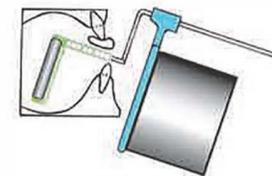


Sensibles Posterior Positioning Guide




Maxillary

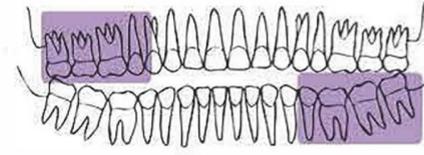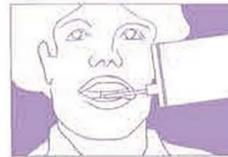


Use either Bite Block, depending on size of sensor.

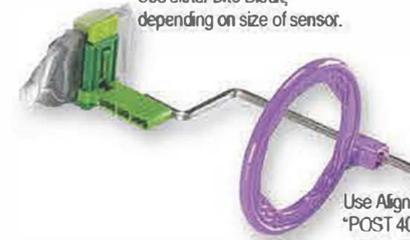

Use Alignment Arm "POST 40858-10."

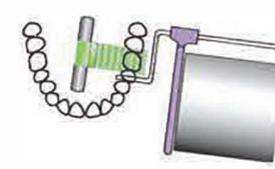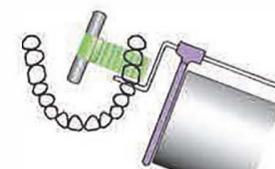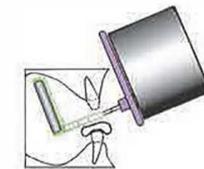




Mandibular

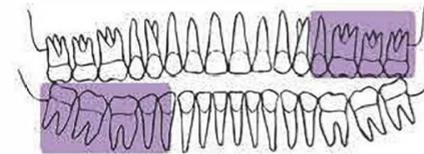



Use either Bite Block, depending on size of sensor.



Use Alignment Arm "POST 40858-10."

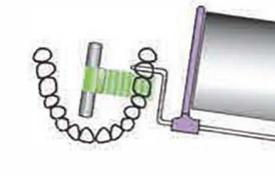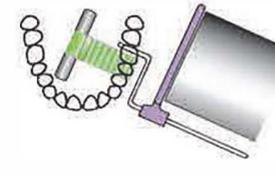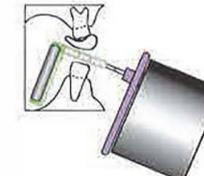



Sensibles Bite-Wing Positioning Guide

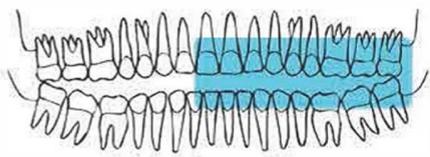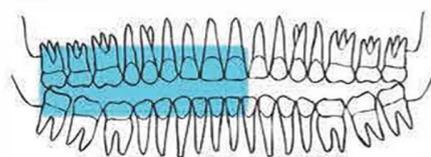

Use either Bite Block, depending on size of sensor, then simply slide Bite Block to mid-line of sensor.



Use Alignment Arm "BITE 40915-10."

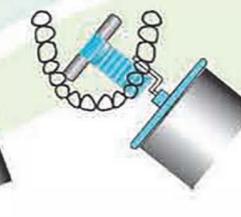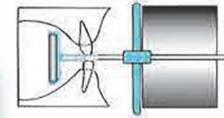

Cleaning and Sterilizing Options (autoclave before first use):

1. Soak for one hour in glutaraldehyde. DO NOT USE PHENOL-BASED GLUTARALDEHYDE. Rinse and dry.
2. Clean ultrasonically, use a general purpose ultrasonic cleaner or detergent/soap and water. DO NOT USE SPECIAL APPLICATION CLEANERS. Rinse and dry.
3. Steam Autoclave (distilled water only). DO NOT CHEMICLAVE OR DRY HEAT STERILIZE. DO NOT USE THE AUTOCLAVE DRYING CYCLE. OR Cold Sterilize. CHEMICALS MAY CAUSE SKIN IRRITATION IN SOME PATIENTS

Training video available here:



Horizontal Imaging



The Medium Sensible holds most size 2, 1 and 0 sensors horizontally.

Vertical Imaging



The Large (two-tone) Sensible holds most size 2 and 1 sensors vertically.

Bite-Wing Imaging



For Bite Wing imaging, simply slide Bite Block to mid-line of sensor.

To Insert Sensor



1. Place sensor in holder



2. Push top down

To Remove Sensor



Push button on back

We Recommend



Large: 80210 Medium: 80200