

7. QUALITY ASSURANCE PROGRAM

Here following the list of the operation required to maintain the continued proper functioning of the unit:

| Frequency | Type of check | Done by | Reference |
|-----------|-------------------------------------|----------------------|---------------|
| Daily | Functioning of the indicator lights | User | Paragraph 7.2 |
| Daily | Laser alignment check | User | Paragraph 7.3 |
| Monthly | Panoramic image quality check | User | Paragraph 7.4 |
| Six-month | 3D image quality check | User | Paragraph 7.5 |
| Yearly | Dosimetry test | Authorized personnel | Paragraph 7.6 |



Note

It is recommended to perform the quality assurance procedures either with the suggested frequency or with the frequency required by local regulations if higher.