



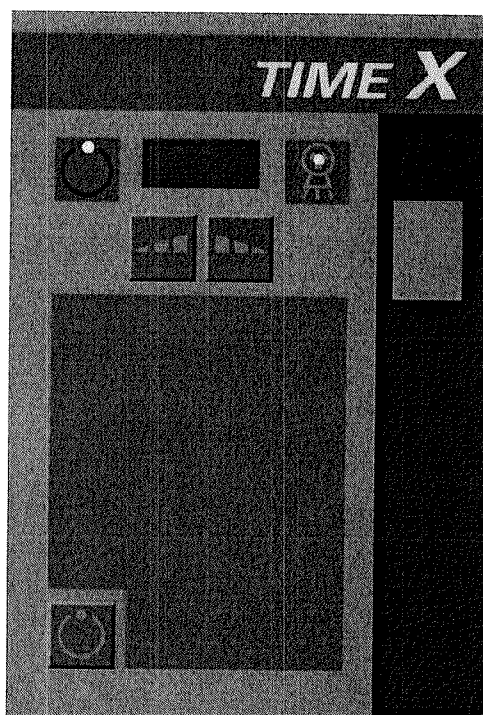

EXPLOR-X 70

with AP TIME X – TIME X timer
(120V version)

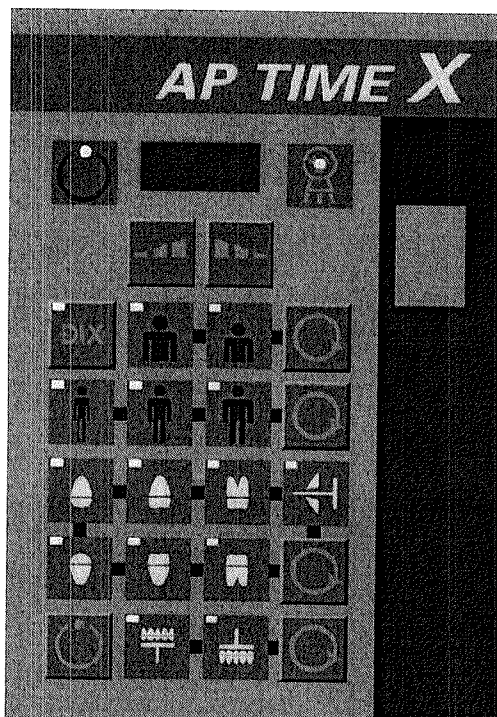

Service manual

10. SPARE PARTS

1 - TIMER: AP Time X - Time X






2 - EXTENSION ARM

3 - SCISSORS ARM

4 - TUBEHEAD

1 - TIMER: AP Time X - Time X

Ref.	Order code	Specifcation	Note
1	5160227400	Rotating pivot spacer	For extension arm
2	6160111500	Support arm	
3	5460130400	Timer cover	
4	4291415900	ON / OFF switch	
5	6660379400	Kit PCB AP TIME X – TIME X	
6	4560311800	Fuse holder	For mobile version
	4560311900	Cap for fuse holder	
7	8160114000	Single stud assy	
8	6260372500	X-ray push button	
9	6660374300	Keyboard assy for AP TIME X	
10	6660374200	Keyboard assy for TIME X	
F2	2300974400	Fuse 6.3x32 10A F	Mounted on Power PCB
F4	2300938000	Fuse 5x20 630mA T	Mounted on Power PCB
FUSE	2300974400	Fuse 6.3x32 10A F	For mobile version

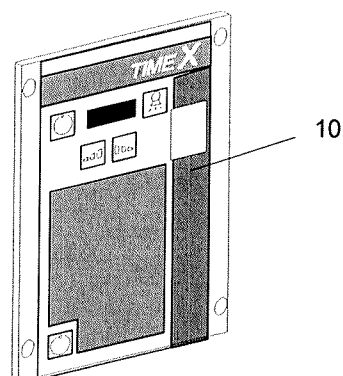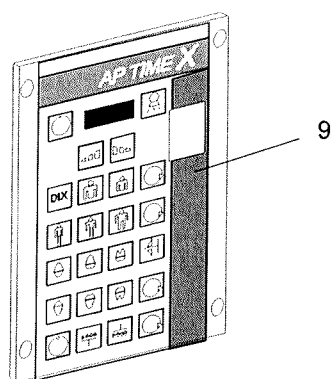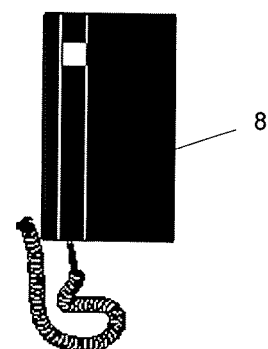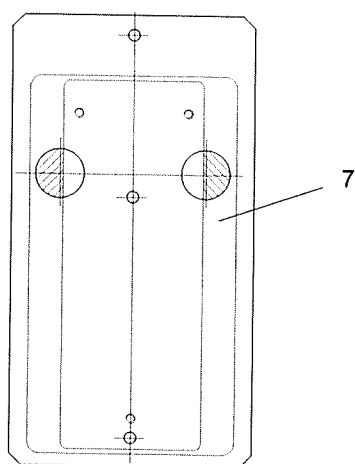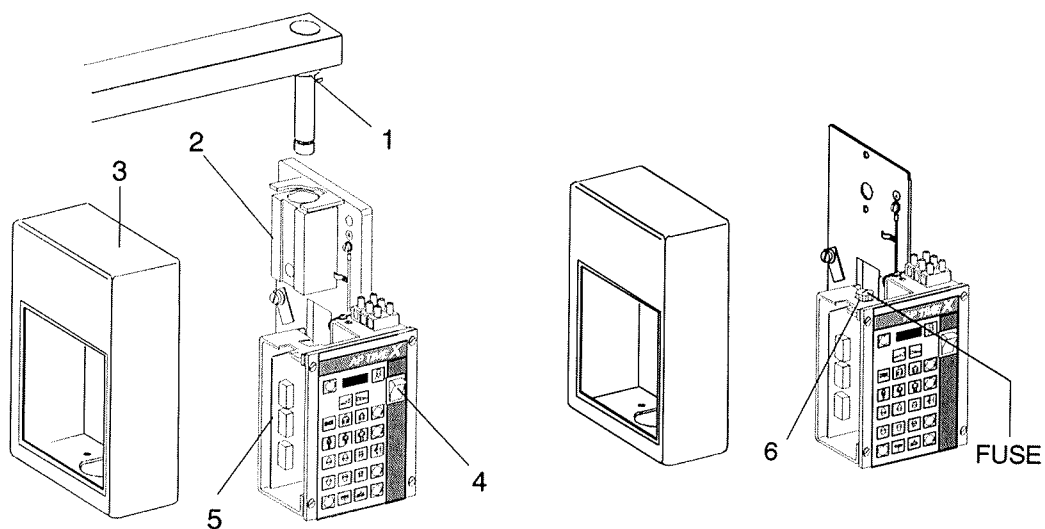


2 - EXTENSION ARM

Oval

Ref.	Order code	Specification	Note
1	5160244200	Cover 75 cm	
	5160244700	Cover 90 cm	
	5160249300	Cover 60 cm	
	5160256100	Cover 30 cm	
2	5160249800	Rotating pivot locking	
3	5260249400	Pivot	
4	6160256000	Busher Ø 25 assy	
5	8160100100	Dental unit support	
7	8260250300	Extension arm 60 cm cpl	
	8260250400	Extension arm 75 cm cpl	
	8260250500	Extension arm 90 cm cpl	
	8260256500	Extension arm 30 cm cpl	

Square

Ref.	Order code	Specification	Note
3	5260201000	Pivot (arm 75, 55, 30 cm)	
	5260205900	Pivot (arm 90 cm)	
5	8160100100	Dental unit support	
6	8160213900	Extension arm 55 cm cpl	
	8160213500	Extension arm 75 cm cpl	
	8160213600	Extension arm 90 cm cpl	
	8160214500	Extension arm 30 cm cpl	

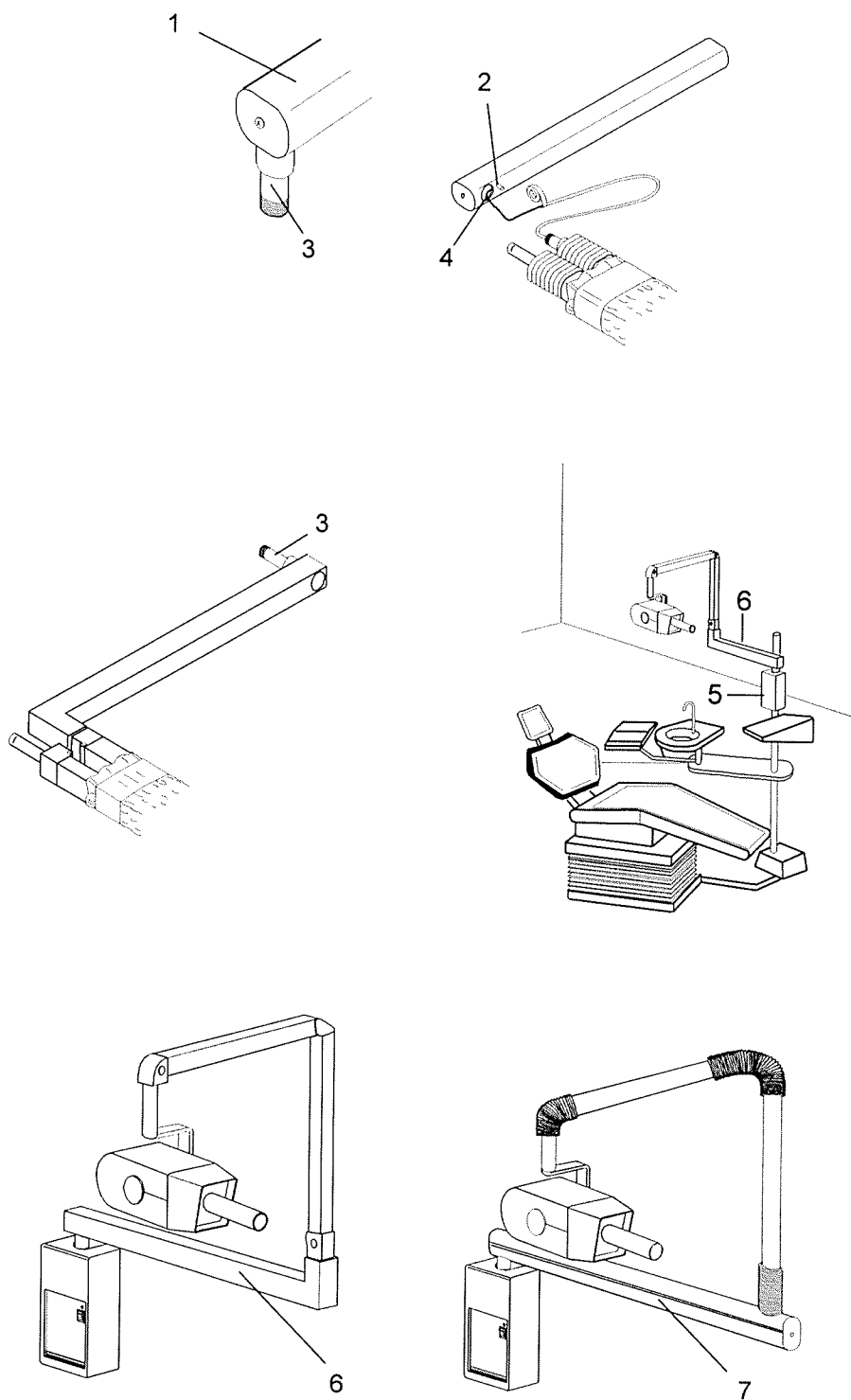

3 - SCISSORS ARM

Oval





Ref.	Order code	Specification	Note
1	5160248100	Scissors arm cover	
2	5460258100	Plastic cap for bellow	
3	5460248300	Bellow (short)	
4	5660248700	Tubehead support cover (metal part)	
5	6160248800	Bellow (central part)	
8	7260209400	Scissor arm cable complete with sliding connector (female)	
10	8160250200	Scissors arm complete	(1)

Square

Ref.	Order code	Specification	Note
6	6660210200	1° scissor arm cover	
7	6660210300	2° scissor arm cover	
8	7260209400	Scissor arm cable complete with sliding connector (female)	
9	5460218800	Central joint plastic cover	
10	8160214000	Scissors arm complete (goffered)	(1)


Note (1):

Please specify where the arm is installed (Mobile, Wall mounted or Dental chair) when ordering.

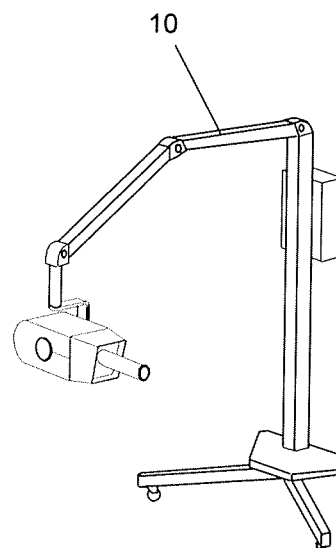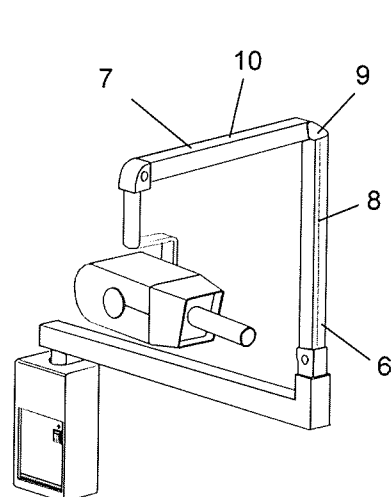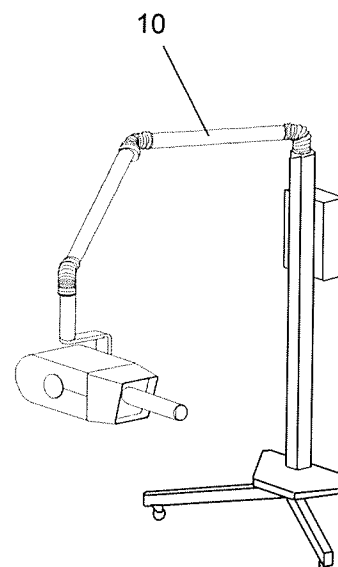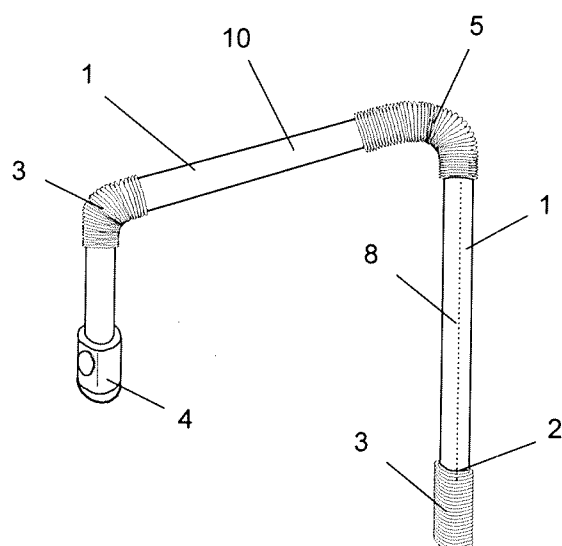

4 - TUBEHEAD

Ref.	Order code	Specifcation	Note
1	5160401800	Aluminium filter	
2	5160423800	Angle indicator	
3	6160405900	Tubehead collimator	
4	6260439600	Sliding connector (male)	
5	6660422300	Explor-X tubehead cover (without label)	
6	7260439500	Tubehead support	
7	8460457000	Tubehead complete assembly	
8	5160402800	Seeger	
9	5460420800	Lateral cover	
10	6160204400	Grounding clip	

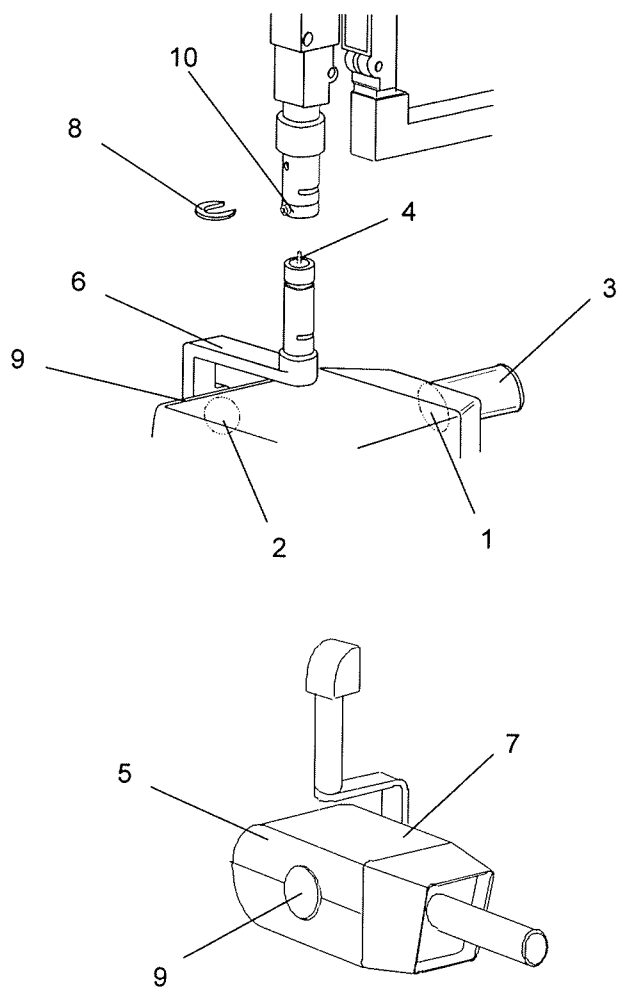

THIS PAGE IS INTENTIONALLY LEFT BLANK

11. FIXING TEMPLATES

The system is equipped with a set of templates, composed of the following elements:

Code	Description
39609122	Template for standard timer version
39609124	Template for remote timer version
39609249	Single stud template